

by anandic

vyaire™  
MEDICAL




Medisorb™ EX and EF EX  
soda lime CO<sub>2</sub> absorbent for GE Carestation™ 600 series

## Medisorb® EX and EF EX absorbent in disposable canisters

Medisorb EX and low alkaline Medisorb EF EX disposable canisters are designed for use with the GE Carestation™ 600 series anesthesia machines to ensure optimal performance.

The EX canisters have been fully tested and validated for use with the GE Carestation 600 series. The pre-filled, sealed canisters can be installed quickly and easily to ensure CO<sub>2</sub> is removed from low-flow, closed loop anesthesia breathing circuits.

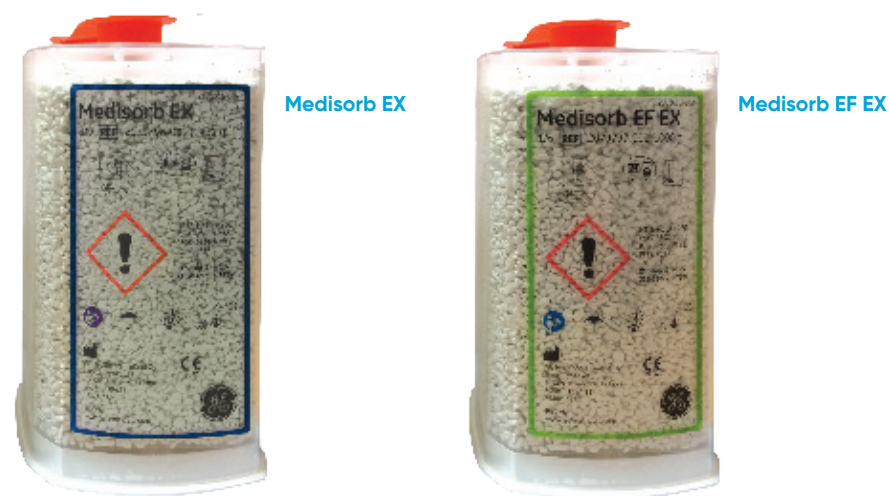

Part no.	Description	Qty. (pkg.)
2079796-001	Medisorb EX	6
2079797-001	Medisorb EF EX	6

### Easy positioning with efficient delivery

Medisorb EX and low alkaline Medisorb EF EX disposable canisters provide easy installation and positioning with the GE Carestation 600 series. Simply place the canister on the tray and use the lever to raise and lock the canister in position.

The disposable canisters are designed to be larger and more efficient, leading to fewer changes, less packaging and lower total cost.

### Color change and reversion

As Medisorb soda lime becomes exhausted, it changes color from white to violet. With the Medisorb EF EX, this color change is longer lasting, thus helping to identify previously used absorbent. The Medisorb EF EX will also turn violet if desiccated. **(Note: Color indication is not a definitive guide to remaining absorbent life and should always be used in conjunction with CO<sub>2</sub> monitoring. Please refer to product IFU for further information.)**

### Compatibility

Disposable Medisorb EX and Medisorb EF EX soda lime has been tested and validated for use with the GE Carestation 600 series for CO<sub>2</sub> removal.

### Conforms to APSF recommendations

During anesthesia administration, anesthetic agent degradation and dry soda lime can cause the production of Compound A and carbon monoxide. Medisorb medical soda lime does not significantly degrade volatile anesthetic when used as described in the absorbent Instructions For Use.

### Simplify through a single source

Purchasing accessories and supplies from a single source means fewer purchase orders to process, vendors to research and contracts to review. Plus, customized supply contracts can save costs. With Vyaire as your single source, you benefit from a dedicated sales and support organization, as well as a global resource and distribution network. Make the right connection with accessories and supplies from Vyaire.

Download important Safety Data Sheets for Disposable Medisorb EX and Medisorb EF EX absorbent for the GE Carestation 600 series at [vyaire.com](http://vyaire.com).





For more information, please contact your local Vyairē representative.

#### GLOBAL HEADQUARTERS

Vyairē Medical, Inc.  
26125 North Riverwoods Blvd  
Mettawa, IL 60045, USA

 Vyairē Medical Oy  
FI-00510 Helsinki, Finland



[vyairē.com](http://vyairē.com)

Trademarks are the property of their Respective owners.

**For U.S., Canada, Europe, Australia, Asia, Middle East, Africa and Latin American distribution.**

© 2018 Vyairē Medical, Inc. or one of its affiliates. All rights reserved. Vyairē and the Vyairē Medical logo are trademarks or registered trademarks of Vyairē Medical, Inc. or one of its affiliates. Medical devices class IIa according to Medical Devices Directive 93/42/EEC. Please read the complete Instructions For Use that come with the devices or follow the instructions on the product labeling. CF/VIS/16/0005